



Quick Start Guide (nurse Call)

- ◇ Make sure the Proxi-Mate unit is plugged in, and / or charged up
- ◇ Make sure the Sensor is properly connected—under bedclothes, on top of mattress
- ◇ Make sure the cables are out of the way and not a trip hazard
- ◇ Make sure the cable is connected to the Nurse Call Point
- ◇ Settle patient / resident into bed / on chair
- ◇ Press the 'On / Off' button—Proxi-Mate Pro shows 'On'
- ◇ When patient / resident rises alarm is triggered. Attend to person then reset Proxi-Mate (if still needed)
- ◇ To RESET press 'On / Off' (Proxi-Mate Pro shows 'Off')

www.proximate.com.au